AXI

Wherever you want 2 go

A close-up photograph of a red exercise machine, likely a rowing machine. The image shows three rows of red perforated seats, each with a black handlebar. The machine's frame is also red, and the background is white. The text "Quality and style" is overlaid in a black box in the upper right corner.

Quality and style

AXI₂GO

Axi transport chair

Are you looking to provide your mobility-impaired visitors with a comfortable, user-friendly chair? Then Axi is the answer! A vandalism and theft-proof transport chair that is just right for your public areas thanks to a unique nesting system. With their contemporary design, these striking Axi transport chairs will boost your image as a modern, service-oriented organisation.

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AXI_{2GO}

Unique nesting system

Transport chairs that are used to take visitors from the car park to the building entrance, for example, can often cause inconvenience. People leave them everywhere except where they belong. To tackle this problem, Axi transport chairs can be stored in an Axi park that can be easily identified by everyone. The chairs can be nested neatly inside each other. The nesting system prevents chaos in your public spaces.

Axi park

The Axi park is easy for everyone to identify.

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A close-up photograph of a modern chair's backrest and seat. The backrest is made of a red perforated metal mesh, while the seat is a yellow perforated metal mesh. The chair's frame consists of smooth, curved bars in red, yellow, and black. A black strap is visible on the left side, attached to the frame. The lighting is bright, highlighting the textures and colors of the materials.

Varied colour range

What colour fits in with your organisation?

The Axi is available in all RAL colours, so you can choose a colour that matches your own house style. Boost your image with the Axi and brighten up your reception area.

The coating

The highly corrosion resistant cathodic coating makes the Axi and all its fastening materials weatherproof.

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Easy to use

Axi in 7 steps

1. If necessary, insert a coin in the coin slot and release the Axi.
2. If present, use the luggage carrier so you can keep your hands free.
3. Start the Axi by pressing the black brake lever downwards.
4. The occupant can now sit down. Before you start, the feed should be on the footrest.
5. Place the walking stick or crutch, if present, in the stick holder.
6. Tackle any obstacles using the tilt pedal.
7. After use, return the Axi to the Axi park and retrieve your coin.

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A close-up photograph of a grey plastic wheel with a black treaded tire. The wheel has a multi-spoke design. The tire has a deep tread pattern. The wheel is mounted on a red vehicle. The background is a plain, light-colored surface.

Safe to use

AXI₂GO

Everything under control

The brake lever engages the Axi brake. As the accompanying person, you therefore have full control over the transport chair. But besides control, stability is also an important factor. Because the wheels are optimally balanced, inexperienced users can use the Axi both indoors and outdoors with no problems at all. The chair is also easy and safe for the occupant to get in and out of.

NB: For safety reasons, Life & Mobility recommends using a seat belt when using the chair outdoors.

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AXI₂GO

Focus on the end user

The user of a transport chair wants to sit comfortably even if they are only using it for a short time. That is why the Axi is fitted with a preformed seat, backrest and armrests. This combination delivers surprisingly good sitting comfort. For even greater comfort, we also offer optional 3D-woven seat and backrest covers.

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A close-up photograph of a red bicycle's front wheel and fork. The wheel has a black tire with a tread pattern and a silver-colored rim with three large, teardrop-shaped cutouts. The black fork is attached to the wheel. The red frame of the bicycle is visible in the background. The text "Built to last" is overlaid in a black box in the upper right corner.

Built to last

AXI₂GO

Durable and vandalism-proof

These days, transport chairs not only have to be durable and low maintenance, but - sadly - they also need to be vandalism-proof. So the frame, seat and backrest of the Axi are made entirely of high-grade steel with a minimum number of moving parts.

The Axi is fitted with maintenance-free, anti-leak tires, so the chair is always light to manoeuvre and easy to steer. Thanks to the protective powder coating, the Axi can be used both indoors and outdoors.

Hard to steal

The optional coin slot and/or anti-theft post significantly reduce the risk of theft.

Tip: You can decorate both the Axi itself and the anti-theft post with a printed design of your own choice.

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