

Seating

Standing

Sleeping

Therapy

Dreama[™] by Jenx

A pressure reducing support system for effective 24hr postural management, suitable for children and adults



Always Supportive...

Dreama™ 24hr positioning system by Jenx

Jenx have long believed that night-time positioning should be of the same high quality as day time positioning; after all, we probably spend more time in our sleeping position than we do in any other! The importance of 24 hour positioning has become widely recognised over the last few years, as the continuation of good positioning throughout the day and night promotes improved skeletal development and better quality of life. We believe that neglecting proper night-time positioning will take away from the benefits of any postural support given during the waking hours.

An Improved Quality of Life:

Dreama™ by Jenx is a key part of any 24hr postural management programme. Recognising that abnormal and posturally destructive tone is present 24/7, Dreama™ system follows on from support offered by conventional positioning. Utilising a variety of **Support and Positioning Pads**, the system can be configured to accommodate even the most complex of user needs.

Supporting at risk limbs and reducing tonal spasms, Dreama™ contributes towards more recuperative periods of rest, which in turn has a positive contribution towards minimising the risk of muscular skeletal deformity. Dreama™ can be used as a preventative system or to reduce the risk of further postural deterioration, and is appropriate for any age.

Users who may benefit from Dreama™ include those with the following medical conditions: Cerebral Palsy, Muscular Dystrophy, Spinal Muscular Atrophy, Arthrogryposis, Rheumatoid Arthritis and associated conditions, Post-trauma, Post-surgery, Spina Bifida and Scoliosis.

INNOVATIVE FEATURE

Flexible Aluminium Base

The base is fixed to the bed using safety straps and the postural support cushions are then clipped into place.

INNOVATIVE FEATURE

Individual Covers

Individual **Covers** mean any that are soiled can be removed and replaced without the need to remove the user from their bed, minimising sleep disruption and the need for assisted transfers.



INNOVATIVE FEATURE

Custom Profiles

The flexible aluminium base contours to create the lying surface of choice for each user, whether it's achieved by placing the base of Dreama™ on a profiling bed or using rolls, wedges or cushions. This means it can be used for a range of different user requirements in a variety of environments (home, care homes, support centres etc.)



INNOVATIVE FEATURE

Customisable and Configurable Positioning

Hundreds of individual specifications can be configured exactly to the user's needs. The extensive range of positioning options is possible because of the innovative modular system that allows the supports to slide easily into channels between the individual cushions. There are up to 16 available channels for the supports meaning that you can achieve exactly the position required.

Always Supportive

Dreama™ 24hr positioning system - providing valuable rest for everyone

“The Dreama™ sleep system gives Oscar a really good sleeping position overnight, which means he gets a better night’s rest and it removes some of our anxiety around ensuring that Oscar maintains the best possible position, but remains comfortable, which means we also get a better night’s rest!”

- Julie, Oscar’s Mum.

Benefits of Dreama™

Better quality sleep or rest due to comfortable and supportive full depth, pressure reducing modular cushions.

Easily contours thanks to strong, flexible aluminium base.

Allows good ventilation between the body and the base, which reduces heat-triggered spasms.

Quiet with no noise or disturbance when adjusting positions because of Glide-lock™ supports.


Exceptional flexibility for 24 hour support with numerous individual accessories.

Very easy to keep clean and hygienic.

Dreama™ by Jenx is a pressure reducing support system for effective 24hr postural management.



Showing Dreama™ in side lying with Ziggy.

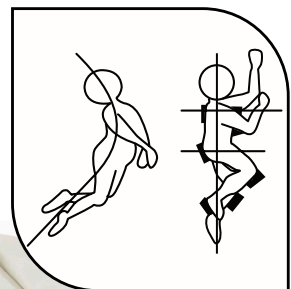

Ziggy by Jenx


This highly adaptable product is square in cross section and can be bent at a number of points to create many different shapes.

Place behind the user in side lying to offer additional support.



Always Innovative

Showing corrected posture with the effective use of Dreama™ support accessories to provide a comfortable nights sleep.

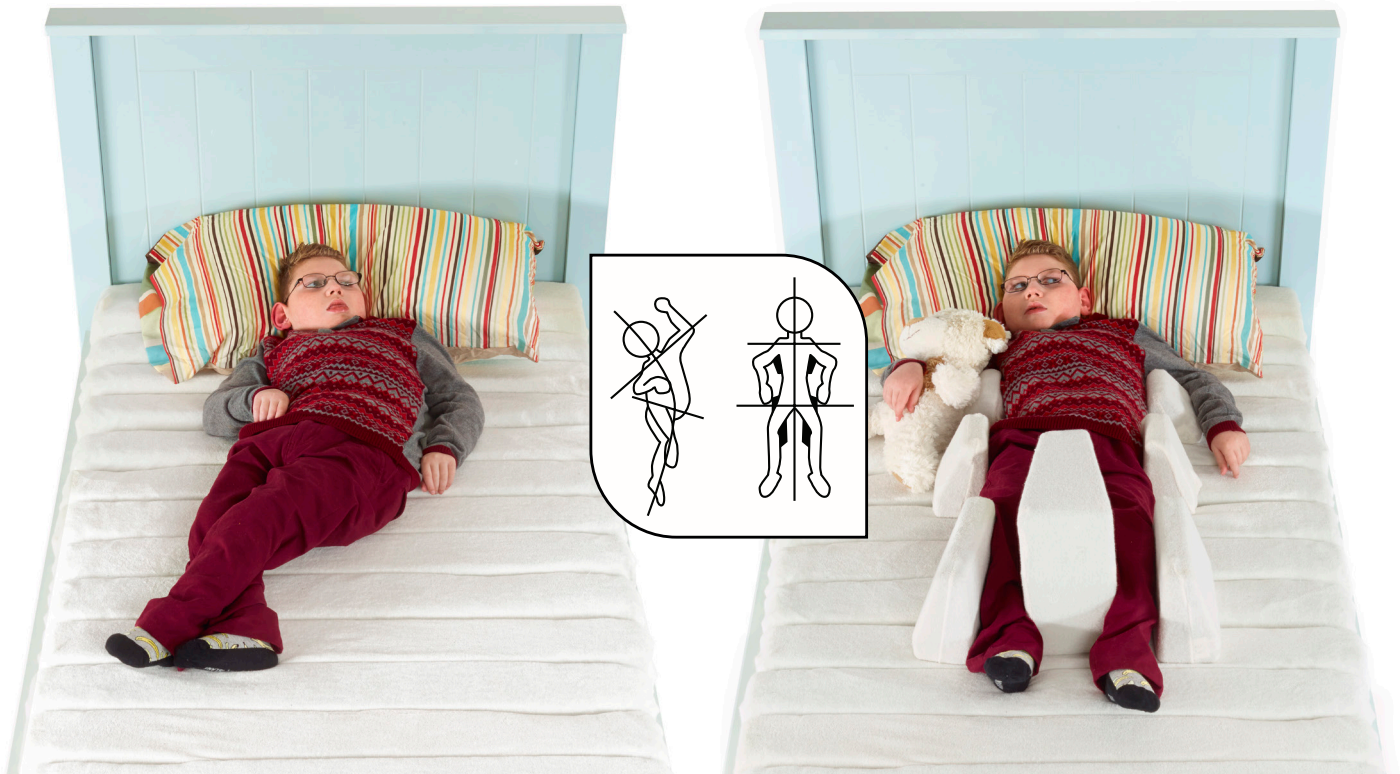

INNOVATIVE FEATURE


Optional Swivel Heads

A number of accessories have the option of a swivel head mechanism that can be used to change the angle of the support. You can change the angle before, during or after positioning the user.

INNOVATIVE FEATURE

Modular Cushions

Each modular cushion is 150mm deep and upholstered in fire-retardant, water resistant, 2-way stretch antibacterial fabric for infection control, as well as being breathable. The spaces between cushions allow airflow, helping to maintain a comfortable temperature. The individual cushions enable specific areas to be cleaned easily during the night with minimum disturbance.

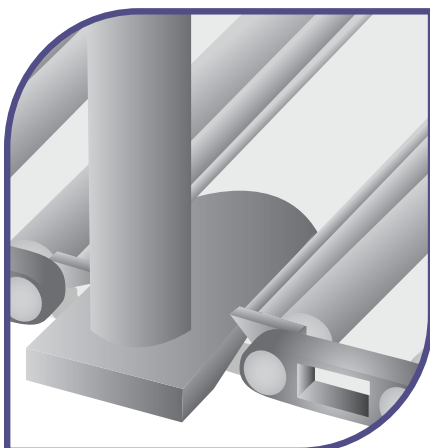

INNOVATIVE FEATURE

Glide-lock™ System

The unique Glide-lock™ system makes positioning simple! The base of the **Support/Positioning Pad** is inserted in to the channel on the base and the pad is gently pushed in to place. Once in place, simply let go and pushing on the pad won't cause it to move. To remove the pad, simply hold it and carefully slide it out. There are also '**Locking Feet**', which can be screwed into place within the channels (they are removable and reusable) either to mark the place a pad should go (making it easier for carers to position a user) or to put behind the pad to prevent users with strong repetitive movements from gradually 'walking' the pad out of place during the night.

Always Thoughtful

Outstanding features and benefits

Pressure Reducing

The sleep system is made from independently-tested pressure-reducing 150mm (6") deep modular cushions. There is no need for any overlays because using any system on top of a pressure reducing base negates the properties of the pressure relieving cushions.

Comfort

Dreama™ is designed to be comfortable. The towelling **Covers** are soft and stretchy and the pressure reducing cushions provide a supportive base. The **Support/Positioning Pads** mean that the user sleeps in a supported position that prevents them from rolling over into an uncomfortable one. Finally, the modular cushion system allows air to circulate, keeping the user cool.

Versatility

Dreama™ offers a full range of positioning opportunities: prone, supine and side lying can all be achieved easily, quickly and quietly.

The individual **Covers** for each **Support/Positioning Pad** and cushion mean that you can always see how each accessory is positioned and micro adjust for the perfect amount of individual support.


Quick & Simple set-up

Dreama™ can be assembled within a few minutes, with the cushions clipping into place on the roll out base via integral clips. Configurations can be changed easily without disturbing the user. Supports can be adjusted or the position changed completely with the user still in bed, even asleep! This means less sleep disturbance for everyone!

The flexible base design means Dreama™ can be used on a profiling bed too.

Lifting and Handling

Dreama™ can be set up with the user in bed. This means transfers can occur with some, all, or no supports in place, reducing the need for lifting. It also makes lifting easier as the position can be adapted once the user is on the bed.

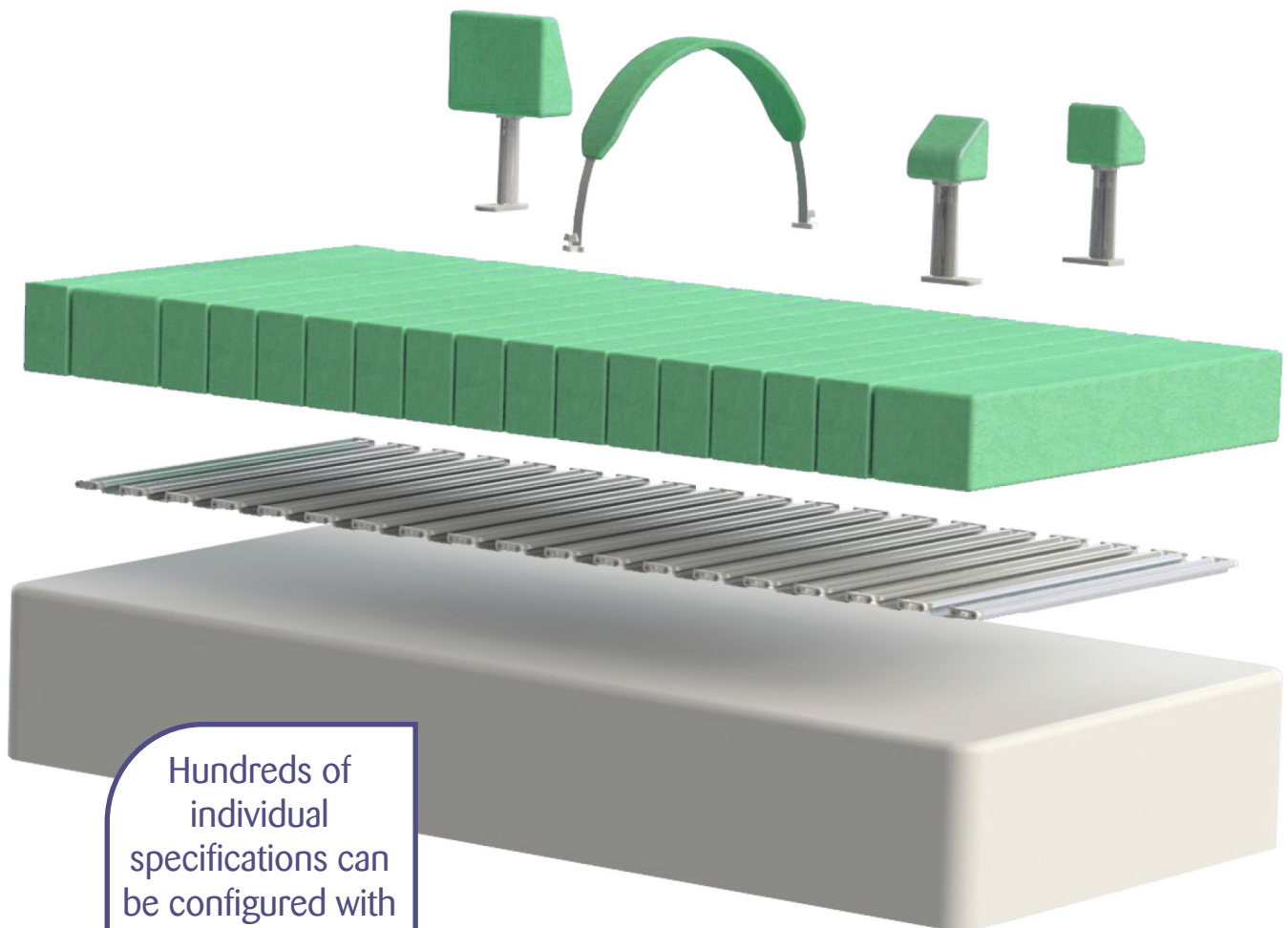

Hundreds of individual specifications can be configured with the wide range of accessories.

Always Configurable Accessories

The wide range of accessories and optional extras mean the Dreama™ by Jenx can be highly customised for every individual's needs.

Positioning Pads

Made from comfortable and hygienic PU and available in 4 sizes. With or without memory foam pad for extra comfort.




Slim Support Pads

Slimline alternative to the **Positioning Pads**. Made from comfortable and hygienic PU. Also available with an optional padded cover.




Abduction Block

This can be used individually or in pairs to ensure hips are abducted and the body is straight. Available in 3 sizes.

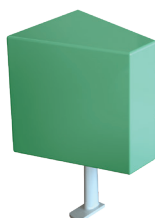


Positioning Hoops

A brilliant and functional accessory, to keep bedding away from the body or allow movement within a limited range for a comfortable night's sleep. Available in 4 sizes.

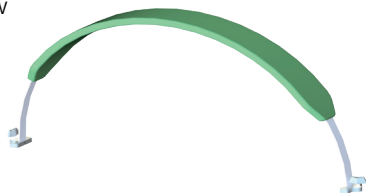



Locking Feet

These useful items are used to mark the position where an accessory should be placed. It keeps accessories in place for users with high movement.

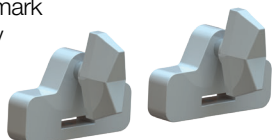

T-Roll

This is height adjustable and helps to support knees, reduce abdominal pressure and prevent scissoring. The sides are height adjustable and the central pommel is removable - a very versatile item.

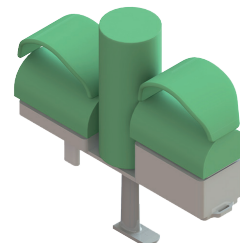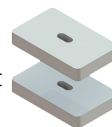

Angle Adjustable Footrest

Ingeniously designed to support feet in supine and prevent dropping.




Knee Cups

Use these to comfortably hold knees in a specified position. Available in 2 sizes.



Side Lying System

This complete system includes combinations of **Positioning Pads, Straps** and **Side Lying (abduction) Foam**. The straps attach to standard **Positioning Pads** with three strap lengths available. The **Side Lying Foam** is also available in 4 sizes.

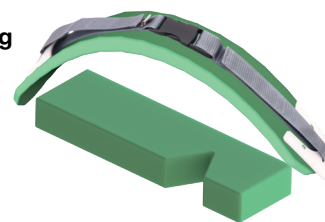

Covers

There are **Covers** available for all Dreama™ accessories. These are provided in white towelling, but can easily be dyed at home to any colour you prefer.



Carry Bags

There are **Carry Bags** available for Dreama™ accessories, modular cushions and base to assist in transporting the system.

Dreama™ by Jenx Technical Specifications

	Measurement	Units	Cot DC06	Cot Bed DC51	Single Bed DC03	Extra Long Single Bed DC15
Age Range	approx.	mths-yrs	9mths - 4yrs	9mths - 7yrs	9mths - Adult	9mths - Adult
User Weight	max	kg	80	100	110	110
Single Cushion Size	length x width x height	mm	560 x 95 x 150	700 x 95 x 150	900 x 95 x 150	900 x 95 x 150
Double Cushion Size	length x width x height	mm	560 x 195 x 150	700 x 195 x 150	900 x 195 x 150	900 x 195 x 150
No. of Cushions Supplied	(single:double)	mm	8:2	10:2	15:2	16:2
Height of System (Inc. Base)	max	mm	170	170	170	170
Footprint	min - max	mm	560 x 1200	700 x 1400	900 x 1900	900 x 2000

Find Your Distributor

To locate one of our carefully selected distributors near you, visit:
jenx.com/where-to-buy



@JenxLtd



@JenxLtd



@JenxLtd



Jenx Limited, Wardsend Road, Sheffield S6 1RQ, United Kingdom
 tel: +44 (0) 114 285 3376
 email: export@jenx.com
www.jenx.com

